Remote Display D-8000 INSTRUCTION MANUAL

1. Table of Contents

2. Safety Notes.....	1	5. Troubleshooting.....	4
3. Part Names.....	2	6. Specifications.....	4
4. Setup.....	2		

2. Safety Notes

Caution Symbols

For optimum performance and safety, please familiarize yourself with the Caution Symbols below. These symbols are designed to alert the user to potential hazards when using this equipment. Ignoring these Caution Symbols may result in serious injury, or damage to the product. Please be sure to review before proceeding with the INSTRUCTION MANUAL.

 **WARNING** This symbol indicates the possibility of serious injury if the product is mishandled or instructions are ignored.

 **CAUTION** This symbol indicates the possibility of physical injury or equipment damage if instructions are ignored.

 This symbol indicates general precautions that should be taken when using this product.

WARNING

- To reduce the risk of electric shock or product damage, never insert or remove the AC adapter with wet hands.
- Do not under any circumstances dismantle or alter the device, as this could result in electric shock or injury as well as adversely affecting the precision of measurement.
- To prevent a fire hazard use only a correctly wired outlet, and do not use a multiple outlet extension cable.

CAUTION

- Do not insert or remove the AC plug by the cable.
- Do not place a finger into any gap or any hole of the equipment. You may be injured.

Maintenance


Please observe the following instructions.

- If the equipment is moved to a place with a temperature difference of 10°C (18°F) or more, leave it for at least two (2) hours before use.
- Avoid subjecting the equipment to excessive shocks or vibrations.
- Never disassemble or adjust the equipment, as this may cause malfunctions.
- When not in use for an extended period of time, unplug the AC adapter from the power outlet.
- Do not wipe the equipment with corrosive chemicals (benzene, acetone, etc.). Please use a neutral detergent to clean the equipment.
- Do not use mobile phones or microwave therapy equipment near this equipment.


3. Part Names

Check Included Items



D-8000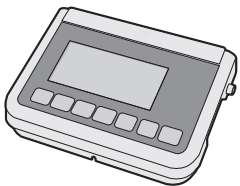

AC adapter

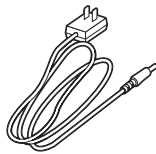
- Instruction (CD or Leaflet)
- Warranty
- Registration Card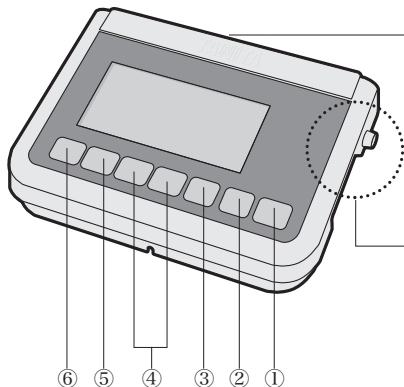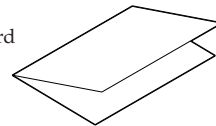


Stand (Rear)

Extend the stand when standing up on a table, etc.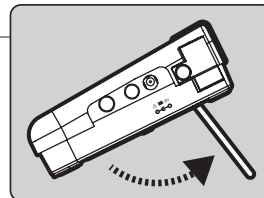

Battery Box

Please use 6 AA alkaline batteries.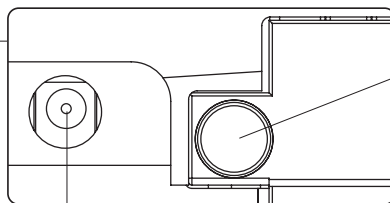


DC Jack Port

Please use the included AC adapter.

<Key Functions>


- ① **ON**Turns unit on.
- ② **OFF**Turns unit off.
- ③ **SET**Sets item.
- ④ **▲ ▼**Used to make selections and enter items.
- ⑤ **FUNCTION**Used to enter the ANT#.
- ⑥ **kg / lb**Switches displayed units.

4. Setup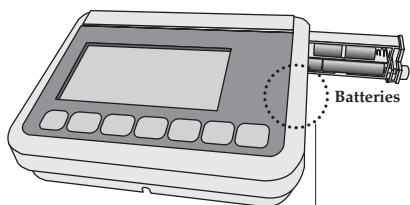


When Using Batteries

- (1) Loosen the display unit battery box screw, and slide the box out.
- (2) Follow the markings on the case and insert 6 AA alkaline batteries.
- (3) Securely close the battery box and tighten the screw.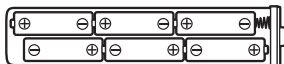

When Using the AC Adapter

- (1) Insert the dedicated AC adapter into the DC jack port of the display unit.
- (2) Plug the AC adapter into a wall outlet.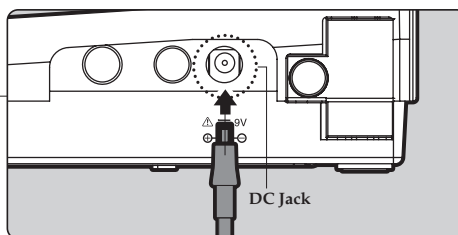


<Setting the Serial Number>

Before proceeding, make sure that the WB-8000RW and D-8000 are within 17 feet (5m) of each other.

The serial number must be set in order for the WB-8000RW and the D-8000 to communicate.

The same assigned serial number should be set for the WB-8000RW as for its corresponding D-8000.


This must be performed when using the device for the first time, or when using the D-8000 with a different WB-8000RW. If it is not performed, measurement values will not be displayed on the D-8000.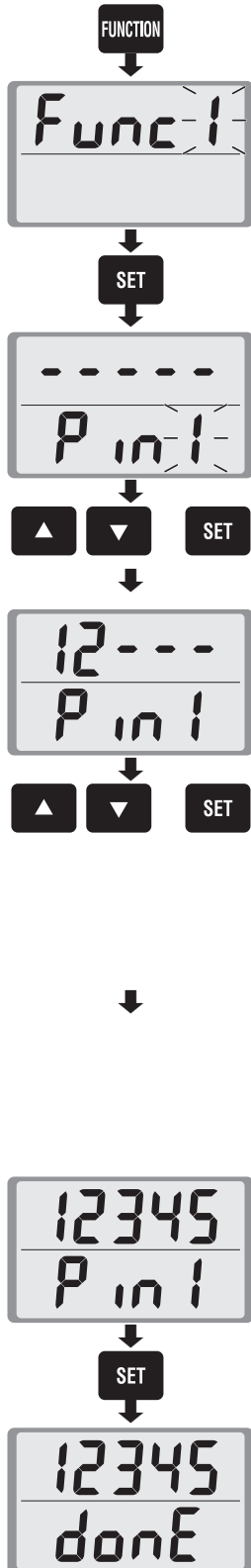

1. Press **FUNCTION** on the D-8000 .
2. When "Func1" appears, press **SET** .

3. "Pin1" will be displayed.◦
[When configuring the WB-8000RW for the first time]
You can register up to 4 WB-8000RWs in advance.
Press **▲ ▼** to select the Pin* number you wish to register (Pin1 to Pin4), and press **SET** .
*Pin : Preset Input Number

4. When the top half of the LCD blinks, press **▲ ▼** to enter the 5 digit number printed on the bottom of the WB-8000RW (see figure below). Press **SET** to move to the next number.

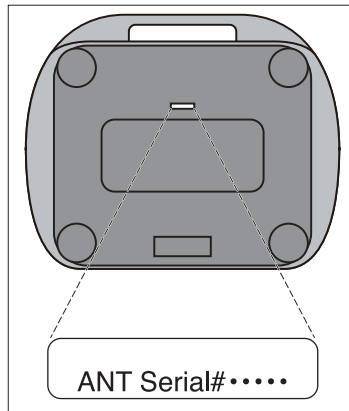5. After entering every digit, press **SET** .

[When configuring a WB-8000RW which has previously been registered]

When the Pin number of a WB-8000RW whose serial number has previously been registered is displayed, the top half of the LCD will show the registered serial number.

Press the **▲ ▼** keys to select the desired Pin number (5 digit serial number), and press **SET** .

Press **SET** .

6. When "done" appears, the configuration process is complete.
 - * To register a different serial number with a different Pin, repeat the process from step 1.
 - * The D-8000 cannot be connected simultaneously to multiple WB-8000RWs. To measure using a different WB-8000RW, set the correct Pin and then perform the measurement.
 - * To register a different serial number for a Pin to which a serial number has already been assigned, overwrite the existing serial number with the new serial number, and press **SET** .

5. Troubleshooting

If you are concerned that the scale may not be functioning correctly, please check the following points before requesting repairs.

Problem	Check Points
"-----" is not displayed when the D-8000 is turned on.	<ul style="list-style-type: none"> • Incorrect Pin input for the device you are trying to use. • The device is off. • The device's mode is set to USB ONLY • The device is not in standby mode
Err C is shown	<ul style="list-style-type: none"> • A transmission error has occurred. Turn the display off.
Err is shown	<ul style="list-style-type: none"> • An error has occurred. Turn the display off.
Lo is shown	<ul style="list-style-type: none"> • Replace the batteries.

6. Specifications

Model		D-8000
Indication	Elements	7 segments - 5 upper, 4 lower
Input	Number of Keys	7 keys
	Height	90cm - 250cm (1cm increments) 3feet - 8feet2inch (0.5inch increments)
Size		159 x 209 x 56 mm 6.3 x 8.2 x 2.2inch
Power Source		AC adapter (included) DC9V 300mA 6 x LR6 - AA Alkaline Battery (Not included)
Power Consumption		0.2W
Temperature Range of usage		5°C to 35°C 41°F to 95°F

USA and Canada

Federal Communications Commission and Canadian ICES Notice

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and Canadian ICES-003. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio or television technician for help.

Modifications

The FCC requires the user to be notified that any changes or modifications made to this device that are not expressly approved by Tanita Corporation may void the user's authority to operate the equipment.

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TANITA® Corporation of America, Inc.
TANITA® Corporation
TANITA Europe B.V.
TANITA Health Equipment H.K.LTD.
TANITA India Private Limited