Radio Wireless Digital Weight Platform  
**WB-8000RW**  
INSTRUCTION MANUAL

**1. Table of Contents**

2. Safety Notes.....	2	5. Operation.....	5
3. Part Names & Specifications.....	3	6. Output data format.....	8
4. Using the WB-8000RW with D-8000 Remote Display.....	4	7. Troubleshooting.....	11

**2. Safety Notes**

**Caution Symbols**

For optimum performance and safety, please familiarize yourself with the Caution Symbols below. These symbols are designed to alert the user to potential hazards when using this equipment. Ignoring these Caution Symbols may result in serious injury, or damage to the product. Please be sure to review before proceeding with the INSTRUCTION MANUAL.

 **WARNING** This symbol indicates the possibility of serious injury if the product is mishandled or instructions are ignored.

 **CAUTION** This symbol indicates the possibility of physical injury or equipment damage if instructions are ignored.

 This symbol indicates general precautions that should be taken when using this product.

 **WARNING**

- To reduce the risk of electric shock or product damage, never insert or remove the AC adapter with wet hands.
- Do not under any circumstances dismantle or alter the device, as this could result in electric shock or injury as well as adversely affecting the precision of measurement.
- To prevent fire hazard use only a correctly wired outlet, and do not use a multiple outlet extension cable.

 **CAUTION**

- Please make sure you place the scale on a level and stable surface. If the equipment is used when the scale is unstable, because not all feet are on the surface, there may be a risk of injury or inaccurate measurement.
- Never jump on the scale, there may be a risk of injury and malfunction of the equipment.
- Do not insert or remove the AC plug by the cable.
- Do not place a finger into any gap or any hole of the equipment. You may be injured.

 **Maintenance**

**This is a precision manufactured and accurately calibrated product. Please observe the following instructions.**

- If the equipment is moved to a place with a temperature difference of 10°C (18°F) or more, leave it for at least two (2) hours before use.
- Avoid subjecting the equipment to excessive shocks or vibrations.
- Never disassemble or adjust the equipment, as this may cause malfunctions.
- When not in use for an extended period of time, unplug the AC adapter from the power outlet.
- Do not wipe the equipment with corrosive chemicals (benzene, acetone, etc.). Please use a neutral detergent to clean the equipment.
- Do not use mobile phones or microwave therapy equipment near this equipment.

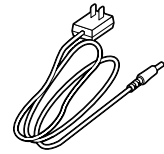
### 3. Part Names & Specifications

#### Check Included Items


WB-8000RW

AC adapter

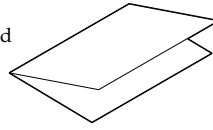Instruction (CD or Leaflet)  
 Warranty  
 Registration Card

#### LED Lamp

Changes color depending on what mode is selected.  
 Indicates status through color and blink pattern.

**This LED lamp is green when the mode selection switch is set to the "ON" position.**

**This LED lamp is blue when the mode selection switch is set to the "USB ONLY" position.**

#### LED off

- This LED is not lit when the mode selection switch is set to the "OFF" position.
- If the LED is off, but the mode selection switch is not set to the "OFF" position, please check the unit power.

#### Blinks once every 4 seconds

- Indicates WB-8000RW is in standby state.  
 \* The unit will be in standby state after it is turned on.

#### Blinks once every 1 second

- Indicates the unit is ready to measure weight. Please step onto the WB-8000RW.

#### Lit

- Indicates that data is being transmitted.

**Blinks red 3 times after turning power on, when blinks red once every 6 seconds**

- Please replace the batteries.

#### Lit Red

- Error  
 \* Automatically recovers.  
 - Make sure to step onto the WB-8000RW only after it has beeped twice or the Green indicator light is blinking.  
 - The unit cannot accurately measure your weight if it detects movement. Please stand on the platform keeping movement to a minimum.  
 - Reading cannot be obtained if the weight capacity is exceeded. Weight capacity is 660lb (300kg).

#### Battery Box (Bottom)

Please use 4 AA alkaline batteries.

#### USB Port

#### DC Jack Port

Please use the included AC adapter.

#### Mode Selection Switch

USB ONLY.....USB bidirectional transmission mode. Use this when using a computer, etc. to control the unit. When this mode is selected, the LED will be blue.  
 OFF .....OFF Mode : Power is off.  
 ON .....ON Mode : Both wireless transmission and USB output (output only) can be used. When this mode is selected, the LED will be green.

#### <Specifications>

Model	WB-8000RW
Measurement System	Strain Gauge Load Cell
Maximum Capacity / Minimum Graduation	300kg / 0.1kg 660lb / 0.2lb
Platform	505 x 415 x 51 mm 19.9 x 16.1 x 2.0 inch
Output Data Interface	USB (B-type)
Power Source	AC adapter (included) DC 9V 200mA
	4 x LR6 - AA Alkaline Battery (Not included)
Power Consumption	60mA
Temperature Range of usage	5°C to 35°C 41°F to 95°F

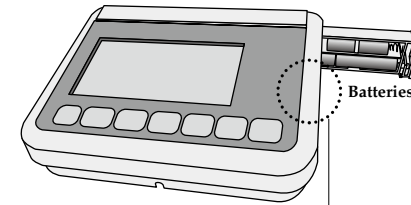
### 4. Using the WB-8000RW with D-8000 Remote Display

#### 1. Provide Power

Provide power to both the D-8000 and the WB-8000RW. Each can be powered by either batteries or the included AC adapter.


\* To ensure a stable and consistent power supply, whenever possible use the included AC adapter.

#### <D-8000 (Optional)>


#### When Using Batteries

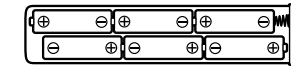- (1) Loosen the display unit battery box screw, and slide the box out.
- (2) Follow the markings on the case and insert 6 AA alkaline batteries.
- (3) Securely close the battery box and tighten the screw.



#### When Using the AC Adapter

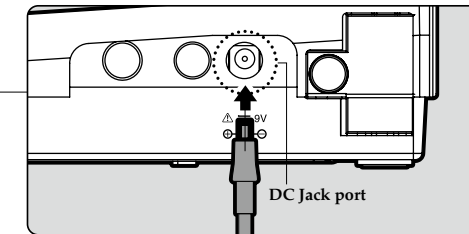- (1) Insert the dedicated AC adapter into the DC jack port of the display unit.
- (2) Plug the AC adapter into a wall outlet.


#### <WB-8000RW>
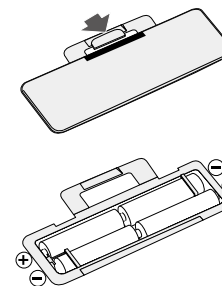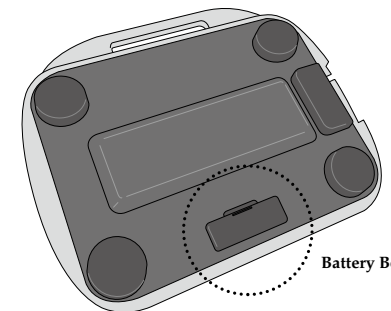
#### When Using Batteries


Open the battery cover on the bottom of the platform. Insert the 4 AA alkaline batteries as indicated.

#### When Using the AC Adapter

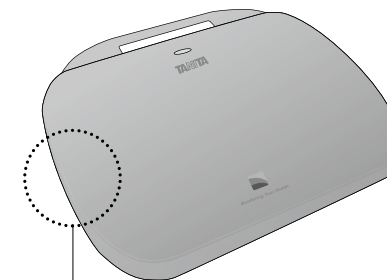- (1) Insert the included AC adapter into the DC jack port on the left side of the weighing unit.
- (2) Plug the AC adapter into a wall outlet.

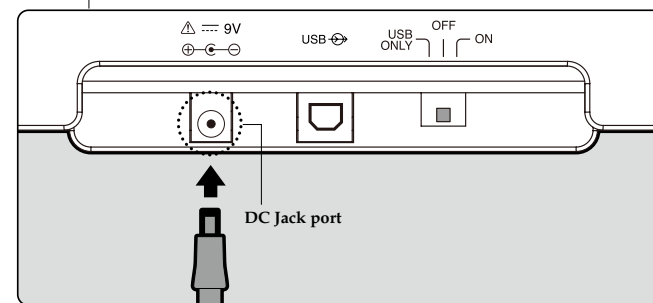**Note:** • Be sure that the polarity of the batteries is set properly. If the batteries are incorrectly positioned, fluid may leak and damage floors. If you do not intend to use this unit for long period of time, it is advisable to remove the batteries before storage.


## 2. Setting the Serial Number

Before proceeding, make sure that the WB-8000RW and D-8000 are within 17 feet (5m) of each other.

The serial number must be set in order for the WB-8000RW and the D-8000 to communicate.


The same assigned serial number should be set for the WB-8000RW as for its corresponding D-8000.

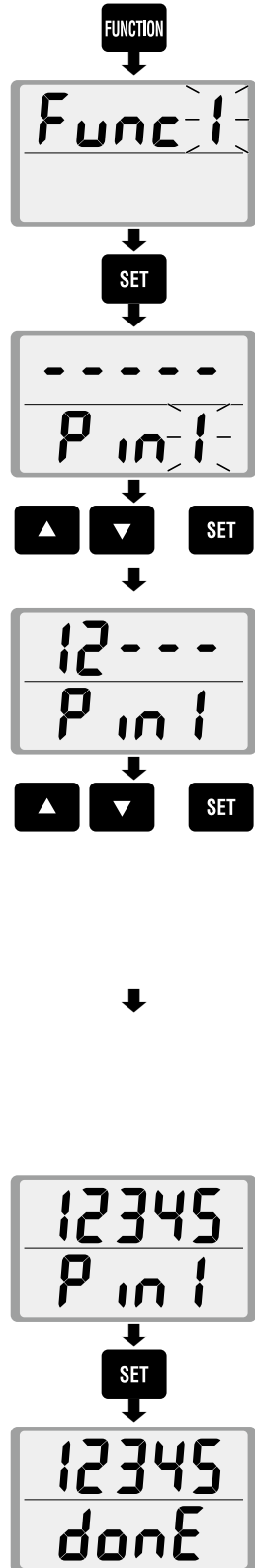This must be performed when using the device for the first time, or when using the D-8000 with a different WB-8000RW. If it is not performed, measurement values will not be displayed on the D-8000.

1. Press the D-8000 **FUNCTION**.
2. When "Func1" appears, press **SET**.
3. "Pin1" will be displayed.  
[When configuring the WB-8000RW for the first time]  
You can register up to 4 WB-8000RWs in advance.  
Press **▲** **▼** to select the Pin\* number you wish to register (Pin1 to Pin4), and press **SET**.  
\*Pin : Preset Input Number

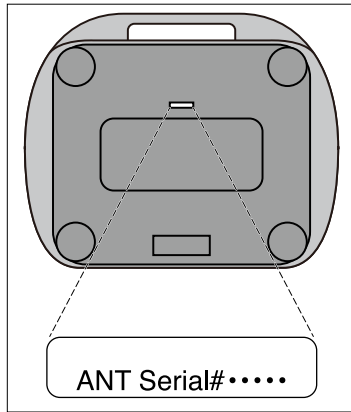4. When the top half of the LCD blinks, press **▲** **▼** to enter the 5 digit number printed on the bottom of the WB-8000RW (see figure below). Press **SET** to move to the next number.

5. After entering every digit, press **SET**.
6. When "done" appears, the configuration process is complete.  
\* To register a different serial number with a different Pin, repeat the process from step 1.  
\* The D-8000 cannot be connected simultaneously to multiple WB-8000RWs. To measure using a different WB-8000RW, set the correct Pin and then perform the measurement.  
\* To register a different serial number for a Pin to which a serial number has already been assigned, overwrite the existing serial number with the new serial number, and press **SET**.

[When configuring a WB-8000RW which has previously been registered]  
When the Pin number of a WB-8000RW whose serial number has previously been registered is displayed, the top half of the LCD will show the registered serial number.


Press the **▲** **▼** keys to select the desired Pin number (5 digit serial number), and press **SET**.

Press **SET**.

## 5. Operation

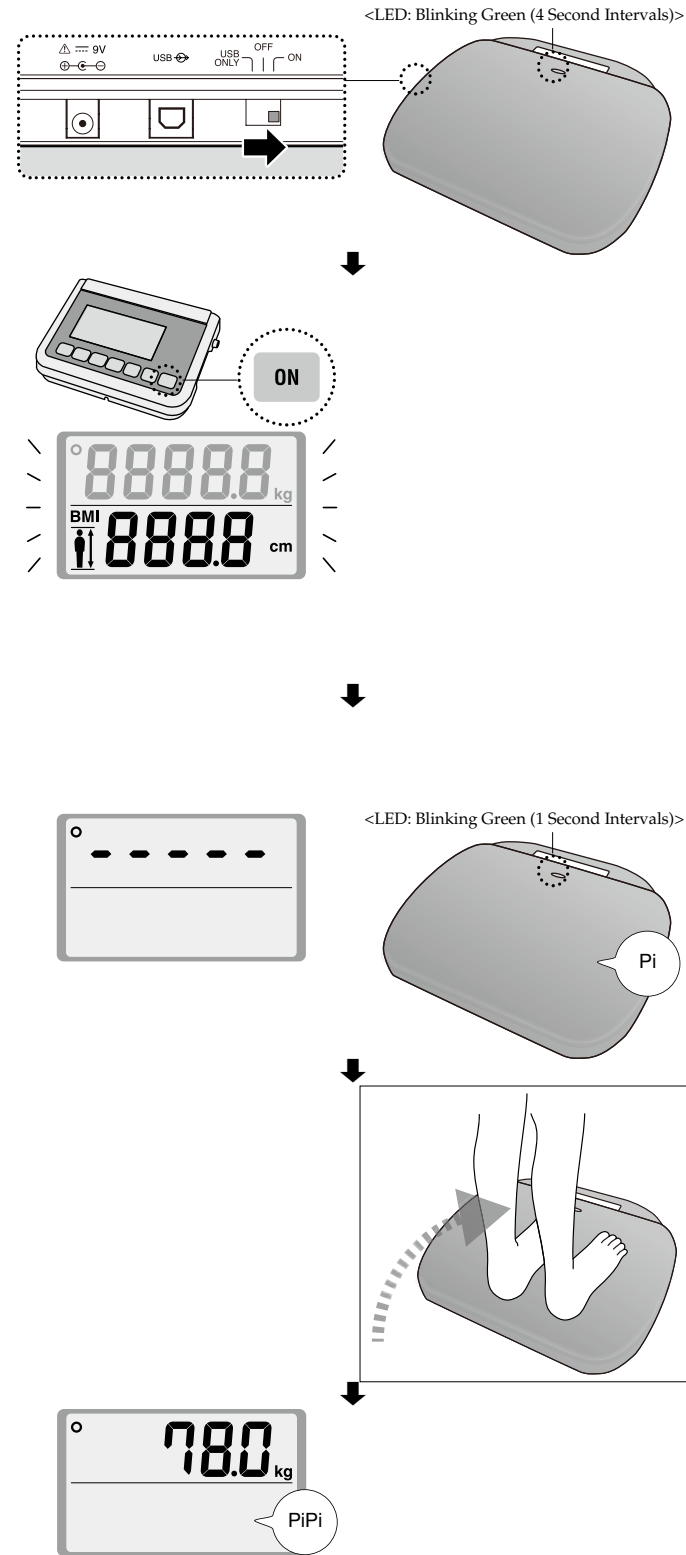Before proceeding, make sure that the WB-8000RW and D-8000 are within 17 feet (5m) of each other.

### 1. Standard Weighing Procedures <weight mode>

- 1) Place the WB-8000RW on a hard, flat surface where there is minimal vibration to ensure safe and accurate measurement.

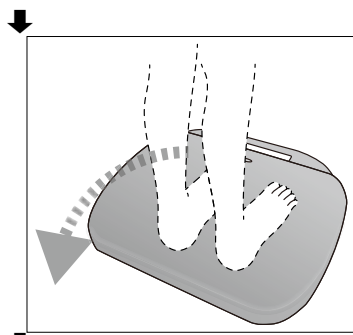
- 2) Set the WB-8000RW switch to the ON position.  
The unit will go into standby mode, and the LED will blink green every 4 seconds.

- 3) Press **ON** on the D-8000.  
The unit will start using the Pin which has been previously set, and will begin communicating with the corresponding WB-8000RW.  
After the unit indicates it is in standby mode, when communication between the two devices is established, "----" will be displayed on the top half of the LCD. The WB-8000RW will start, and it will beep and switch to measurement mode (the LED will blink green in 1 second intervals).


- Note:**
- Pressing and holding **FUNCTION** for 2 or more seconds will cause the currently selected Pin to be displayed.
  - To set or change the Pin, see Setting the Serial Number page in the Setup section.
  - When the D-8000 cannot communicate with the paired WB-8000RW, the display will indicate "ErrC", and the D-8000 power will turn off.

- 4) Step onto the WB-8000RW while the Green Indicator Light is blinking on every second.

After the measurement is complete, the platform will beep twice, the Green Indicator light on longer blinks (remains solid), and weight will automatically be displayed on D-8000.
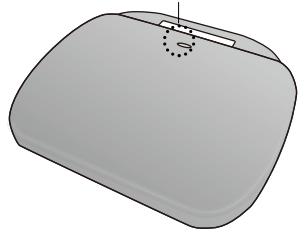
<LED: Blinking Green (4 Second Intervals)>

5) Carefully step off WB-8000RW after measurement is complete.

**Note:** • If you do not step off the scale within 30 seconds of weighing, the WB-8000RW LED will be lit red and emit two short beeps to remind you to step off the scale.

When data output ends, the WB-8000RW LED will turn off, and when you step off the WB-8000RW, it will switch to standby mode (the LED will blink green in 4 second intervals).

6) When the D-8000 display disappears, the scale can be used for weighing again.

The value shown on the D-8000 will disappear, and after switching to standby display, the unit will beep, and "-----" will be shown on the display.

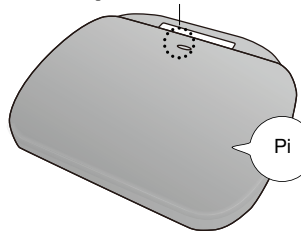
When the WB-8000RW starts up, beeps, and switches to measurement mode (the LED will blink green in 1 second intervals), please step onto the WB-8000RW.

**Note:** • If no measurement signal is received by the D-8000 from the WB-8000RW for a certain amount of time, the D-8000 display value will disappear. When this happens, the WB-8000RW will switch to standby mode (the LED will blink green in 4 second intervals).


\* To end measurement, press the D-8000 OFF key to turn off the display. Switch the WB-8000RW mode selection switch to the OFF position.


<LED: Blinking Green (1 Second Intervals)>


Measurement Ended
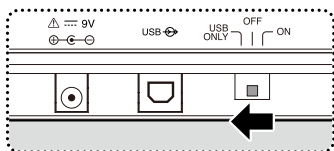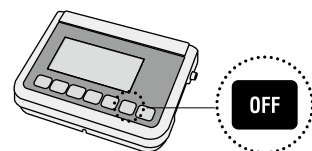
<LED: Blinking Green (4 Second Intervals)>

## 2. BMI mode

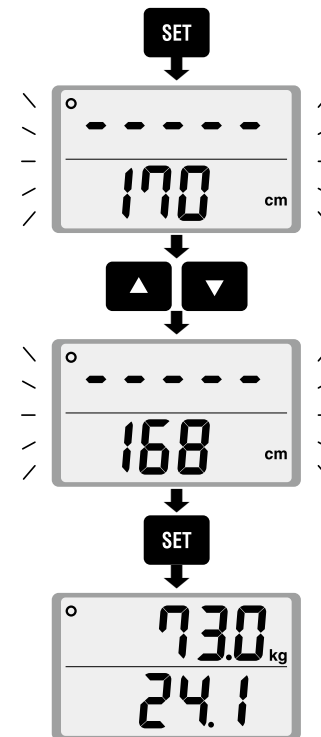1) After step 3 "Press **ON** on the D-8000" of the "Standard Weighing Procedures", the top half of the LCD will display "-----". Press **SET**.  
The bottom half of the LCD will blink "170" (when the unit is set to "kg").

Press **▲ ▼** to enter your height in cm.

**Note:** • When the weight display unit is set to "pounds", enter your height in inches.


2) Once you have finished entering your height, press **SET**.

3) Step onto the WB-8000RW.

**Note:** • Enter your height within 30 seconds of the D-8000 displaying "-----".

4) After weighing, the D-8000 will display your weight and BMI.

## 3. Changing Units

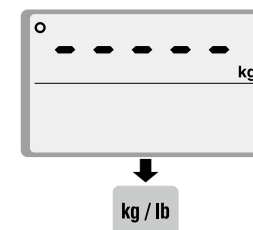
You can change units by pressing the kg/lb key when the D-8000 displays "-----".

**Note:** • This function may not be available for all models.

## 4. Output Function

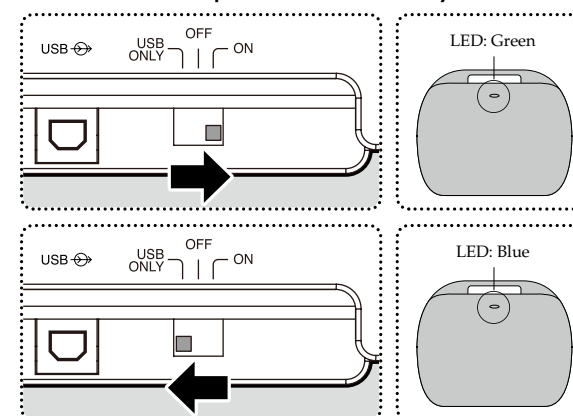The WB-8000RW's output function is controlled by the mode selection switch's position.

• When the WB-8000RW's mode selection switch is set to ON, both wireless and USB output can be used. USB transmission will be set to output-only, and when the display is set your weight and BMI will be automatically output. The LED will be green.

• When the WB-8000RW's mode selection switch is set to USB ONLY, only USB output can be used. USB transmission will be possible in both directions (input and output). The LED will be blue.

## 6. Output data format

The section covers the exporting of data from the scale to an external device (e.g. PC) using a USB compliant signal.

### Configuration

Communications standard	USB
Communications method	Asynchronous transaction
Signal speed	9600 baud
Data bit length	8 bits
Parity	None
Stop bit	1 bit
Terminator	CR+LF

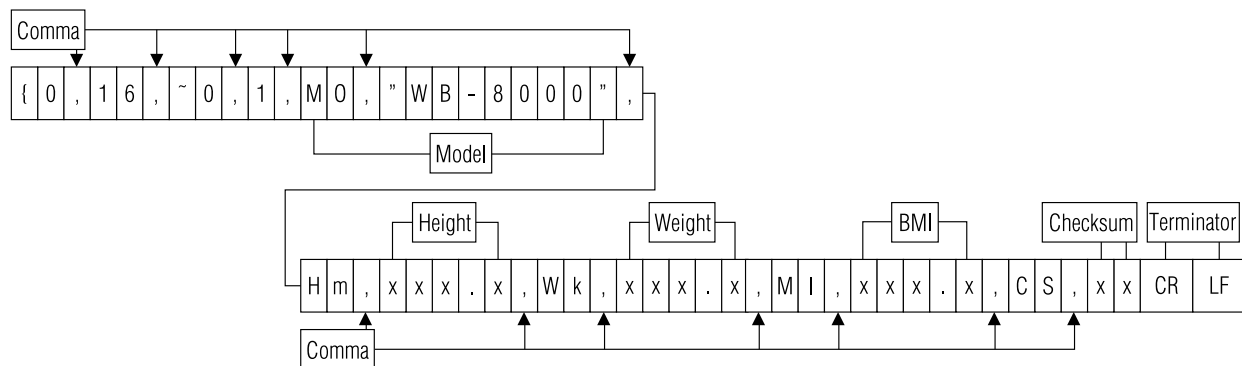
- Note:**
- USB connector (B - Type 4 - pin female) is located on the side of the WB-8000RW.
  - Please provide your own cable as necessary as none are included. USB cable: A - Type 4 - pin (male) - B - Type 4 - pin (male)
  - You must install the necessary driver onto your PC, available download from <http://www.tanita.com>.

### Output

Name of item	Header	Output data (ASCII code)		Pattern	
				BMI mode	Weight mode
Control data	{0	Fix to 16	2 Bytes fixed	✓	✓
Control data	~0	Fix to 1	1 Byte fixed	✓	✓
Model	MO	"WB-8000"	9 Byte fixed	✓	✓
Height	Hm	xxx.x	4 - 5 bytes	✓	—
Weight	Wk	xxx.x	3 - 5 bytes	✓	✓
BMI	MI	xxx.x	3 - 5 bytes	✓	—
Checksum	CS	xx	2 Bytes fixed	✓	✓

- Note:**
- The data is divided with commas (,) for each data.
  - The terminator (end of the data) is CR (ASCII code 0DH), LF (ASCII code 0AH).
  - The control data for the items 1 - 4 are for expansion. They are not used presently so the receiving side can ignore them.
  - Because of its specifications, the scale will output an irrelevant signal approximately 0.1 second after data output. Make sure you ignore this signal and do not import it.

### [Output example]

- When the WB-8000RW's Mode Selection Switch is set to ON, both wireless output and USB output can be used. USB transmission will be set to output-only, and when the display is set your weight and BMI will be automatically output. The LED will be green.
- When the WB-8000RW's Mode Selection Switch is set to USB ONLY, only USB output can be used. USB transmission will be possible in both directions (input and output). The LED will be blue.

### <Data Format when Mode Selection Switch is Set to USB ONLY>

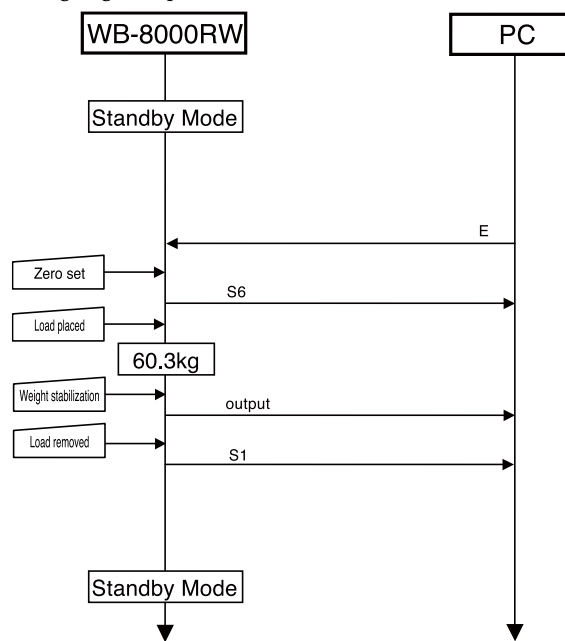
- This mode is used to control the WB-8000RW via computer.
- This scale is not capable of receiving instructions from an external device. When using the ON mode.

**Note:**

- USB connector (B - Type 4 - pin female) is located on the side of the WB-8000RW.
- Please provide your own cable as necessary as none are included. USB cable: A - Type 4 - pin (male) - B - Type 4 - pin (male)
- You must install the necessary driver onto your PC, available download from <http://www.tanita.com>.

### <Weighing Example>

- 1) Use the mode selection switch to switch to USB ONLY position.
- 2) Ready to communicate to computer.  
The unit will switch to "State1".
- 3) An [E] command will be received from the computer.
- 4) The product will begin performing zero setting, and switch to "State5".  
When zero setting is complete, it will switch to "State6".  
\* When the unit switches to "State6", it will send [S6] to the computer.
- 5) Step on the weighing platform. When the weight measurement stabilizes, the unit will switch to "State7", and the weighing results will be sent to the computer.  
\* Please see the separately listed output format.
- 6) When you step off the weighing platform, the unit will switch to the "State1" state.  
\* When the unit switches to "State1", it will send [S1] to the computer.
- 7) The device will switch to standby mode.

### <List of Valid Commands>

- \* In the chart, "Response" indicates that a response is sent in reply to a command. The contents indicated in the "Response" section of the chart are sent before the action listed in "Action" is performed.
- \* A [CR][LF] is input after the "Message" and "Response" items in the chart.
- \* After "Command" in the chart, send a [CR][LF].
- \* When a command other than those listed in the chart is received, [!][CR][LF] will be sent, indicating that an invalid command has been received.

State Confirmation		State0	State1	State2	State5	State6	State7
Command	Response	—	✓	✓	✓	✓	✓
S?	None	Outputs current state of device				SX	

X is a 1 byte character  
 Message States  
 S0: "State0" (Not USB bidirectional transmission mode)  
 S1: "State1" (Configuration has not been completed)  
 S2: "State2" (Configuration has been completed)  
 S5: "State5" (Setting zero)  
 S6: "State6" (Zero set, awaiting load)  
 S7: "State7" (Weighing output has been completed, no load on device)

State Confirmation		State0	State1	State2	State5	State6	State7
Command	Response	—	✓	✓	—	—	—
W?	None	Outputs program version.				W8000XXXX	

"XXXX" section will contain program version

State Confirmation		State0	State1	State2	State5	State6	State7
Command	Response	—	✓	✓	—	—	—
D3XXX	None	Set height.				D3, Hm, XXX.0	

X is a 1 byte character  
 Sets height.

Start Weighing		State0	State1	State2	State5	State6	State7
		—	✓	✓	—	—	—
Command	Response	Action			Message		
E	None	Setting zero (changes to "State5")			None		
		Zero is set, awaiting load (changes to "State6")			S6		
		When load stabilizes, measurement results are output (changes to "State7")			None		
		Load drops to 2.0kg or less (changes to "State1")			S1		

Start BMI Measurement		State0	State1	State2	State5	State6	State7
		—	—	✓	—	—	—
Command	Response	Action			Message		
G	None	Check if height has been set			When set	None	
					When not set	E4	
		Setting zero (changes to "State5")			None		
		Zero is set, awaiting load (changes to "State6")			S6		
		When load stabilizes, measurement results are output (changes to "State7")			None		
Load drops to 2.0kg or less (changes to "State1")			S1				

Reset		State0	State1	State2	State5	State6	State7
		—	✓	✓	✓	✓	✓
Command	Response	Action			Message		
Q	@	Reset (changes to "State1")			None		

Stop Weighing		State0	State1	State2	State5	State6	State7
		—	✓	✓	✓	✓	✓
Command	Response	Action			Message		
q	@	Stop weighing (changes to "State1" or "State2")			None		

Height value set before weighing is stop is retained.

#### <Error Commands>

E0	Weighing error
E1	Weight limit exceeded
E4	Start BMI measurement command sent without height being set
E6	Set height value exceeds input parameters. BMI under 6.0

## 7. Troubleshooting

If you are concerned that the scale may not be functioning correctly, please check the following points before requesting repairs.

Problem	Check Points
"-----" is not displayed when the D-8000 is turned on.	<ul style="list-style-type: none"> <li>• Incorrect Pin input for the device you are trying to use.</li> <li>• The WB-8000RW is off.</li> <li>• The WB-8000RW's mode is set to USB ONLY</li> <li>• The WB-8000RW is not in standby mode</li> </ul>
The displayed weight is not accurate	<ul style="list-style-type: none"> <li>• The WB-8000RW is not in standby mode</li> <li>• There is interference</li> <li>• The WB-8000RW has been installed incorrectly</li> </ul>
The kg/lb button does not function	<ul style="list-style-type: none"> <li>• Please contact a sales office</li> </ul>
Err C is shown on D-8000	<ul style="list-style-type: none"> <li>• A transmission error has occurred. Turn the display off.</li> </ul>
Err is shown on D-8000	<ul style="list-style-type: none"> <li>• An error has occurred. Turn the display off.</li> </ul>
Lo is shown on D-8000	<ul style="list-style-type: none"> <li>• Replace the batteries.</li> </ul>

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USA and Canada

##### Federal Communications Commission and Canadian ICES Notice

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules and Canadian ICES-003. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio or television technician for help.

##### Modifications

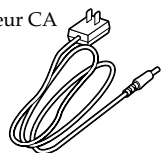
The FCC requires the user to be notified that any changes or modifications made to this device that are not expressly approved by Tanita Corporation may void the user's authority to operate the equipment.

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### **TANITA Europe B.V.**
### **TANITA Health Equipment H.K.LTD.**
### **TANITA India Private Limited**